texinov

TEXISHIELD® CH

MASQUE CHIRURGICAL SURGICAL MASK - CHIRURGISCHE MASKE Type II R (EN 14683:2019 + AC:2019)

1 2 3 4

5

6

MD - A

LOT - B

C - C

REF - D

E - E

i - F

G - G

H - H

CE - I

NON STERILE - J

K - K

L - L

LATEX - M

www.texinov.com

MDB TEXINOV

56 route de Ferrossière 38110 Saint-Didier-de-la-Tour
France

Notice TexiShield® CH Masque médical Type II R - V1.0 - 21/03/2022

FR

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MASQUE MÉDICAL (TYPE II R)

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INSTRUCTION FOR USE**MEDICAL MASK (TYPE II R)**

- Bacterial Filtration Efficiency (BFE) $\geq 98\%$
 Respiratory Resistance $\Delta P < 60 \text{ Pa/cm}^2$
 Microbial cleanliness $\leq 30 \text{ CFU/g}$ max
 Splash resistance pressure $\geq 16 \text{ kPa}$

DESCRIPTION

Medical mask (also called care mask, hygiene mask or surgical mask) made of polypropylene with nose clip.

The CE marking affixed to this article signifies compliance with the global requirements in terms of security and performances of regulation (EU) 2017/745 concerning medical devices. Medical masks are class I MDs.

REFERENCES

References	Color
TexiShield® CH-A03	White
TexiShield® CH-B02	Blue
TexiShield® CH-C02	Green

INDICATION

Single-use medical mask for adults.

Worn to cover the mouth, nose and chin, the medical mask provides a physical barrier to reduce the risk of spreading infections, especially in epidemic or pandemic situations.

CONTRAINDICATION

Any use not described in the indications. Wearing a mask is contraindicated for individuals with respiratory failure.

LIMITATIONS OF USE

These masks are hygiene masks intended to protect others against aerosols emitted by the wearer. They comply with the requirements of European standard EN 14683:2019 + AC:2019 Type II R and are resistant to splashes.

WEARING THE MASK

- 1/ Wash your hands before touching the mask.
- 2/ Place the mask on the face so that the text "Made in France" is positioned on the outside.
- 3/ Put the elastics behind the ears.
- 4/ Bend the nose clip to fit the shape of the nose to prevent unfiltered air from entering and pull it under the chin to finalise the adjustment.
- 5/ Avoid readjusting or touching the front of the mask while wearing the mask.
- 6/ Dispose of the mask immediately after use, preferably in a rubbish bin with a lid.

MAINTENANCE

None. The mask is for single use. It must be disposed of as intended after a single use.

STORAGE AND TRANSPORTATION

Always store in a clean, dry place away from direct sunlight and out of the reach of children. Storage temperatures between 5°C to 40°C (41°F to 104°F). Humidity rate <70%.

This item should be transported as it has been supplied by the manufacturer.

WARNINGS

Before using a mask, consult an occupational physician or an occupational safety officer to ensure that the mask will perform its intended function. This mask does not provide oxygen. Use only in well-ventilated areas with sufficient oxygen (above 17%). Do not use it when toxic substances in the air constitute an immediate danger to your health or life.

Do not use a mask if the expiry date has expired.

Discard the mask if its damaged.

Never leave a child alone with these masks.

Any serious incident in connection with the device should be notified to the manufacturer and to the competent authority of the Member State in which the user and / or patient is established.

SYMBOLS

A - Medical device	B - Batch number
C - Legal manufacturer	D - Reference
E - Expiry date	F - Consult recommendations
G - Store away from direct sunlight	H - Store away from moisture
I - This device complies with European regulations. CE marking	J - Non-sterile device
K - Single-use device	L - Storage or preservation temperature between 5°C and 40°C (41°F and 104°F)

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