**DESCRIPTION / INDICATION**

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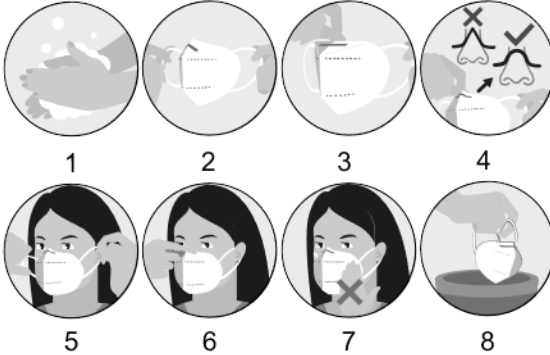
Respiratory protective masks FFP2 NR for single use from MDB Texinov are filtering half-mask to protect against particles, they meet the requirements of the standard EN 149:2001+A1:2009. These masks protect against liquid or solid sprays (dust, smokes, aqueous aerosols...). They do not protect against gas and vapour.

**KONTRAINDIKATION**

Any use not described in the indications. Wearing a mask is contraindicated for individuals with respiratory failure.

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- 1/ Wash your hands before touching the mask.
- 2/ Take a mask.
- 3/ Unfold the nose clip until the fold is removed.
- 4/ Round the nose clip as shown in the illustration
- 5/ Position the mask so that it covers the face correctly (from the nose to the chin) and attach the elastics behind the ears.
- 6/ Adjust the shape of the nose clip on the nose by keeping the nose clip round. To ensure that the respiratory mask tightly adheres to your face, cup your hands over the mask to block up its surface of filtration as much as possible without moving it and breathe out strongly. If any air flows around your nose, readjust the nose bar. In case tightness obtained is not satisfactory, do not enter the area where it is required to wear the mask. Choose a different model of mask to obtain the required tightness.
- 7/ Avoid readjusting or touching the mask while it is being worn.

8/ Remove the mask with the elastics without touching the front and dispose of it immediately after use, preferably in a rubbish bin with a lid.

**WARTUNG**

This mask does not require any maintenance, it is single use. It has been designed for one single continuous work session i.e. 8 hours. It must be discarded after a single use.

**WARNUNGS**

Any failure to comply with use and adjustment controls and warnings concerning the use of the respiratory mask may reduce its efficiency. The reuse of a mask already used may cause risks of contamination of the wearer and people nearby due to the deterioration of the performances of the mask.

Use only in well-ventilated areas with sufficient oxygen (above 17%). Do not use it when the content of toxic substances constitutes an immediate danger to your health or life.

Do not use in explosive atmosphere

Never leave a child alone with these masks.

It's unlikely that the requirements for tightness are met if beard's hair go through the face seal.

Leave the premises and dispose the mask if:

- You feel dizzy or feel any pain
- It becomes hard to breath and you feel oxygen is missing
- Air quality is compromised (contaminants)
- The mask is damaged / deteriorated
- The mask seems to hamper the breathe
- The mask is tainted by blood or any other infectious substance.

Before using a mask, consult an occupational physician or an occupational safety officer to ensure that the mask will perform its intended function.

Any serious incident in connection with the device should be notified to the manufacturer and to the competent authority of the Member State in which the user and / or patient is established.

**STORAGE AND TRANSPORTATION**

Always store in a clean, dry place away from direct sunlight and out of the reach of children according to the recommendations on the packaging (between 5 to 40°C and below 85% of humidity). This item should be transported as it has been supplied by the manufacturer. Do not use masks that have passed their expiry date. Expiry date is indicated on the packaging.

**PROTECTION CLASS PERFORMANCE**

|                            |                                   |            |
|----------------------------|-----------------------------------|------------|
|                            | Paraffine oil initial penetration | ≤ 6 %      |
| <b>Regulation. (EU)</b>    | NaCl initial penetration          | ≤ 6 %      |
| <b>PPE 2016/425</b>        | Breathe resistance at 30L/min     | ≤ 0.7 mbar |
| <b>EN 149:2001+A1:2009</b> | Breathe resistance at 95 L/min    | ≤ 2.4 mbar |
|                            | Breathe resistance at 160 L/min   | ≤ 3.0 mbar |

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Notified body implicated into the EU Type Examination (Module B and D) according to PPE regulation 2016/425/EU: Apave Exploitation France SAS (n°0082) - 6 Rue du Général Audran - 92412 COURBEVOIE cedex - France.

The internet address where the EU declaration of conformity can be assessed is the following: [www.texinov.com](http://www.texinov.com)

**SYMBOLE**

|                                   |   |
|-----------------------------------|---|
| A - Batch number                  | B - Reference   |
| C - Manufacturer                  | D - Expiry date   |
| E - Store away from light and sun | F - Store away from moisture  |
| G - Consult recommendations       | H - This device complies with European regulations. CE marking                |
| I - Single-use device             | J - Storage or preservation temperature between 5°C and 40°C (41°F and 104°F) |
| K - Latex-free                    | L - Non reusable  |

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