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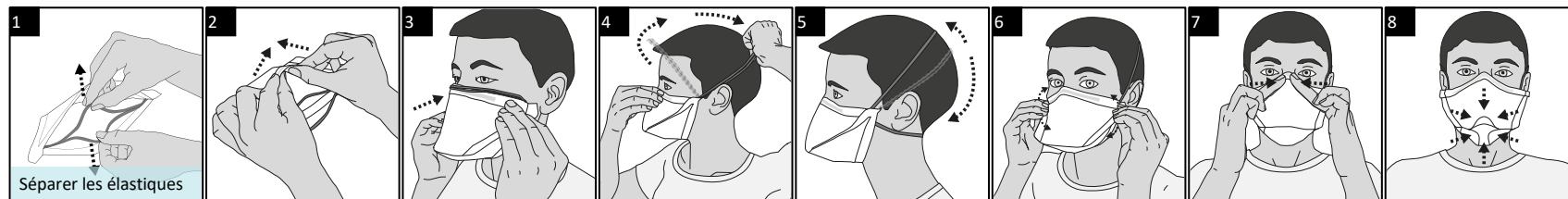
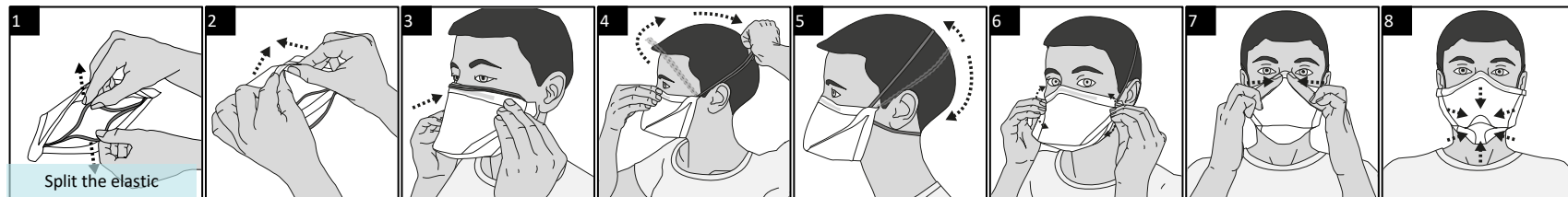
TexiShield® – Item references: F2-B / F2-B02 / F2-B-NW / F2-B02-NW

Instructions for use of respiratory masks FFP2 NR D for single use

1. Description and indication

Respiratory masks FFP2 NR D for single use from MDB Texinov are filtering half-masks to protect against particles, they meet the requirements of the standard EN 149:2001+A1:2009. These masks protect against liquid or solid sprays (dust, smokes, aqueous aerosols...). They do not protect against gas and vapour.

The TexiShield® FFP2 NR D range is composed as follows:

References	Size	Packaging
F2-B	M	Yes – individually wrapped
F2-B02	L	Yes – individually wrapped
F2-B-NW	M	Not individually wrapped
F2-B02-NW	L	Not individually wrapped

Figure 1

References for size L are recommended for faces where the distance between the top of the nose and the bottom of the chin is greater than or equal to 120 mm (see figure 1).

Composition: polypropylene, spunbond and meltblown, nose clip with metal wires, elastomer fasteners.

2. Control before use

This mask is suitable for people trained for its use. Check the storage expiration date on the packaging. Don't use any mask with an expired storage date. Take the mask out of its pack at the time of using it only. Before using it, wash your hands and ensure that it is intact and that the elasticised strings, that split in two parts, are intact too. If the mask is damaged, put it throw.

To ensure that the respiratory mask tightly adheres to your face, cup your hands over the mask to block up its surface of filtration as much as possible without moving it and breathe out strongly. If any air flows around your nose, readjust the nose bar. If any air flows around the mask, carefully adjust the strings to the back of the head. When you breathe in, the mask should deform.

In case tightness obtained is not satisfactory, do not enter the area where it is required to wear the mask. Choose a different model of mask to obtain the required tightness.

3. Use

Follow instructions 1→2→3→4→5→6→7→8

This mask does not require any maintenance. It has been designed for one single continuous work session i.e. 8 hours. Any failure to comply with use and adjustment controls and warnings concerning

the use of the respiratory mask for the whole period of exposure, may reduce its efficiency.

This mask is for single use.

The reuse of a mask already used may cause risks of contamination of the vector and people nearby because of the deterioration of the performances of the mask.

It's unlikely that the requirements for tightness are met if beard's hair go through the face seal.

- Leave the premises and dispose the mask if:
- You feel dizzy or feel any pain
- It becomes hard to breath: oxygene is missing
- Air quality is compromised (contaminants)
- It is damaged
- It seems to hamper the breathe
- It is tainted by blood or any other infectious substance.
- Dispose of the mask according to the current regulation.
- Do not use in explosive atmosphere.

4. Storage and Expiry date

Storage according to the recommendations on the packaging. (Between 5 to 40°C and below 85% of humidity).

The product expires 5 years after its date of manufacture. Expiry date is indicated on the mask and on the packaging.

5. Performance justifying the protection class

This filtrating half-mask meets the protection requirements defined by the regulation PPE 2016/425, EN 149:2001+A1:2009.

Regulation. (EU) PPE 2016/425 EN149:2001+A1:2009	Paraffine oil initial penetration	≤ 6 %
	NaCl initial penetration	≤ 6 %
	Breathe resistance at 30L/min	≤ 0,7 mbar
	Breathe resistance at 95 L/min	≤ 2,4 mbar
	Breathe resistance at 160 L/min	≤ 3 mbar

6. Details of the certification laboratory

Notified body implicated into the EU Type Examination (Module B and D) according to regulation PPE 2016/425/EU: Apave Exploitation France SAS (n°:0082) - 6 Rue du Général Audran - 92412 COURBEVOIE cedex - France.

The internet address where the EU declaration of conformity can be assessed is: www.texinov.com **CE MARK 0082**

7. Meaning of symbols

	Manufacturer		Maximum relative moisture of storage conditions
	Protect from light and sunlight		Temperature range of storage conditions

	End of storage date
	Batch number
	For single use
D	Resistant to Dolomite clogging
	Recyclable

	Product reference
	Read instructions for use
NR	Non reusable
	Latex free Graphene free Phthalate free Bisphenol free

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